

506 FOLD AWAY GRAB RAIL RANGE



506A



506B



506D

Introduction

The Days Healthcare 506 range of fold away grab rails are made from powder coated steel tube with a base plate to secure it to the wall. All models in this range have the support rail hinged at the plate allowing it to be folded away and locked into an upright position. The 506D model has a height adjustable leg which is required when extra support is needed.

Directions for use

The mounting plate has mounting holes to secure it to the wall. It should be fitted by a competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user.

Leg adjustment (506D only)

To adjust the height, grasp the 'Snap Lock' between forefinger and thumb and pull it out. Slide the leg out to the required position and push the clip firmly back into the hole. Leg adjusts with 8 x 25 mm (1") increments. Leg adjustment 725-925 mm (29½ -36½"). **NOTE:** Set the leg and rail height before securing the mounting plate to the wall.



Snap Lock

Maintenance

The Days Healthcare 506 grab rails are designed for minimum maintenance and require the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives or bleach based cleaners which may cause damage or discolouration. Periodically check the wall plate fittings and the pivot point of the rail are secure.

Specification

Model No.	Length	Max. User Weight
506A	750 mm	114 Kg
506B	550 mm	80 Kg
506D	740 mm	125 Kg

Warranty Information

The Days Healthcare model no. 506 grab rails are warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur. The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non observance of the instructions set out above.

