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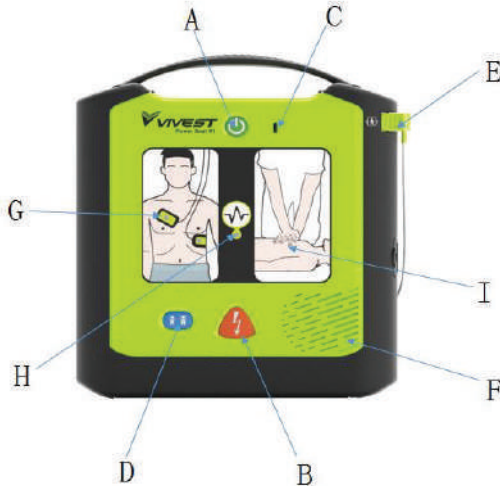
A: Power Button - In standby mode, press this button to enter rescue mode; In rescue mode, press the button for at least 3 seconds to shut the device down and return to the standby mode.

B: Shock Button - This button will flash when the charge is completed and can be pressed to deliver defibrillation energy to patient.

C: Status Indicator - A flashing green light indicates that the device is normal. A flashing red light indicates that the device is faulty.

D: Pediatric Button - If you need to enter the pediatric mode, press this button, and the device will prompt "To enter the Child mode, press the child button for 3 seconds". Then press this button for 3 seconds, and the device will switch to the pediatric mode. (If you need to switch from pediatric mode to adult mode, shut down and restart the device.)

E: Pads Cable Connector - The pads cable connector (Hereafter calls "pads connector") will be pre-connected with the host.

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F: Speaker - Send voice prompts or Beep sounds.

G: Pads Indicator - This light is always on when the device is activated but the pads are not attached to the patient's chest or poor adhesion.

H: Heart Rhythm Analysis Indicator - When the device is analyzing heart rhythm, or charging or waiting for the shock to release, the light is turned on, indicating "Don't touch the patient."

I: CPR Indicator - When the device enters the CPR stage, the light is always on.

J: Quick Operation Guide - Simple introduction to the basic steps.

K: Device Label - The device label includes the device identification number, etc.

L: USB Interface - It is used to export the data, assist software upgrade and set up parameters (For service personnel only).

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Check the symptoms of the patient first, if unconscious, unresponsiveness and no breathing or abnormal breathing, please follow the steps below:

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1. Call the emergency center:

Determine whether the patient is unconscious, unresponsiveness and no breathing or abnormal breathing. If so, do the following steps.

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2. Press the power button to turn on the device:

Once turned on, the device will run power-on self-test automatically.

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3. Determine the patient type (adult/pediatric).

If the patient type is pediatric, press the pediatric button to activate the pediatric energy.

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4. Remove the clothing from the patient's chest.

5. Attach the pads to patient according to the figure instruction.

6. Operate according to the voice or text

prompt. The device will charge to the rated energy when the shockable rhythm is detected.

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7. Press the shock button if there is a shock

advice. Don't touch the patient and those accessories or device connected to the patient during defibrillation.

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STEP 1

Press the green power button to turn on the AED.

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
STEP 2

Take out the electrode package from the back of the AED, tear off the electrodes, and attach to the patient's chest properly (As shown in the figure below).

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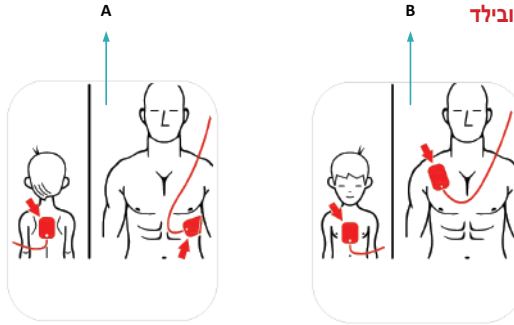STEP 3

The AED will analyze the heart rhythm automatically, and no one is allowed to touch the patient at this moment.

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Caution: The AED will not analyze heart rhythm automatically if the electrodes are not attached properly.

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STEP 4

Carry out the follow-up operations according to the result of heart rhythm analysis.

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- If shock instructed, press the shock button;
- Otherwise, the AED would enter the CPR stage automatically.

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Caution: No one is allowed to touch the patient while defibrillating.

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