

CA355L SHOWER CHAIR MANUAL

Assemble instructions:




Component:

1. Seat 1 PC
2. Foot..... 2 PCS
3. Backrest..... 1 PC
4. Backrest tube 2 PCS
5. Armrest..... 2 PCS (Left& Right)

Spare parts:


- ① SCREW M6*40MM.....5 PCS
- ② SCREW M6*35MM.....4 PCS
- ③ SPACER.....5 PCS

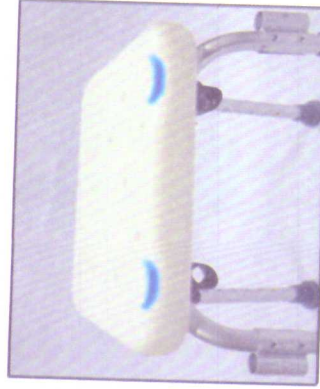


3. Overturn the seat, please note the side with sinking position is the front side.




1. Put the foot tube into the groove of seat as picture shows;

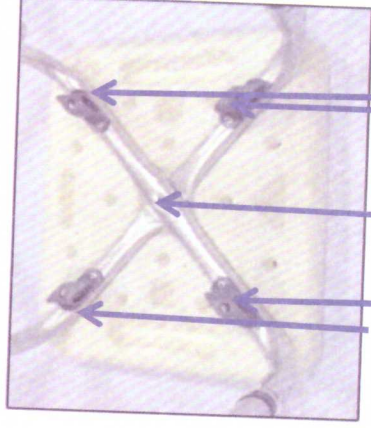


2. Put the other tube as picture shows, assemble the tube in the middle with screw ① and spacer; then assemble the other 4 points with screw ①.

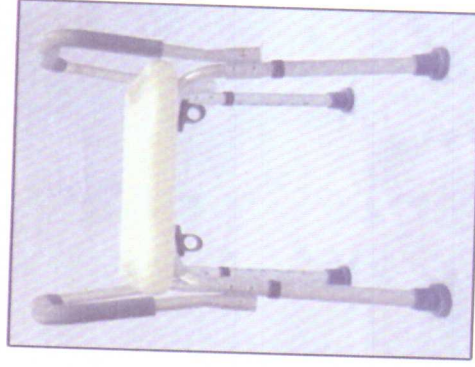

4. Put the armrest into the connect part of feet, make sure the spring pin is out.

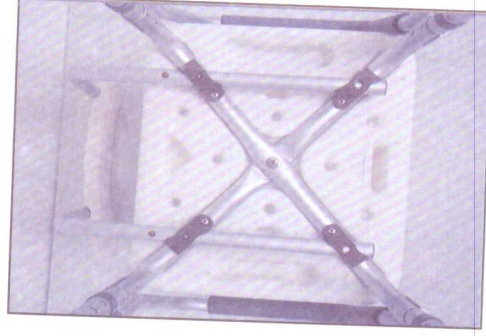

5. Assemble the backrest and backrest tube with 4 screws ② and spacers, then put the backrest into the bottom of seat as picture shows.

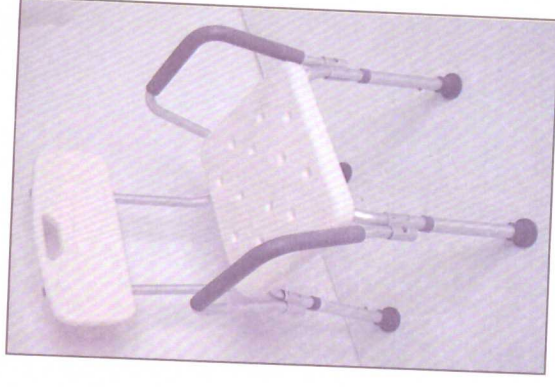

Finished

Safety and maintenance:

1. Before use, check degree of tightness of all parts and stability of frame, do not overload 100 KGS;
2. Clean it frequently and disinfect in periodic time, place it in dry and ventilated environment.